Additional information as to page 25: Cervical mucus

The following cervical mucus consistencies can now also be recorded:

∅ Nothing seen, nothing felt

t Nothing seen, dry feeling


f Moist feeling, no mucus visible

S Mucus is thick, white, yellow, lumpy or creamy

S+ Mucus is glassy, transparent, resembling raw egg white, flexible, thread-like, liquid, red or flows away like water, wet feeling, slippery vaginal introitus

S+H is the peak for cervical mucus. The characteristics correspond to those for S+. The input S+H is only possible on one day per cycle and only during the fertile days. This day is the last day with the best mucus quality [S+]. S+H is determined retrospectively and therefore only recorded on the following day when the change to lower cervical mucus quality has taken place.

The information is displayed in a daily chart and in the cycle curve.

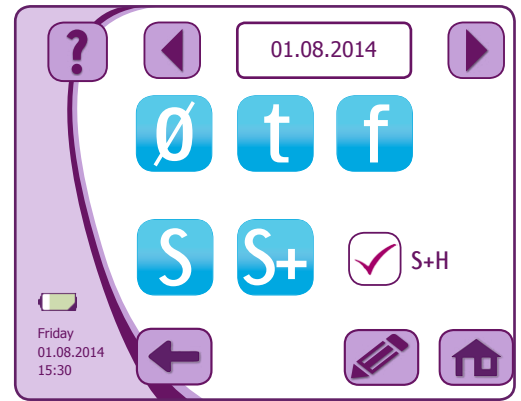Replaces text on page 36: Date/time

Attention: Any change to the date or time can influence your cycle evaluation. Please note the information on the device. It is not possible to set the date to a time in the past.

It is essential to note: The measurement time window is automatically adjusted to the newly set time and cannot be changed within a cycle. The current cycle no longer becomes inactive automatically.

Replaces text on page 49: Time difference/travel

When travelling to a different time zone, you should pause your measurements until you have acclimatised to your destination [at least 3 days for travel outside Europe].

It is possible to change the time without your current cycle becoming inactive. If you have any questions, please contact the cyclotest® service hotline in good time.

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